




Description	<p>Hygienic Cover for bedpan and commode chair with a super-absorbent pad Absorption capacity: 700 ml Cover dimensions : 24.5" x 18" (62.2 x 45.7 cm) No expiry date Eco-friendly product Made in Canada </p>
Item Number	PG1019
Presentation	Hygienic cover® is a bedpan/commode liner containing a super-absorbent pad which solidifies bodily fluids and reduces odors. It is used in association with the Hy21® bedpan support or with a standard bedpan or commode.
Cover dimensions	Length – 24.5" (62.2cm). Width – 18" (45.7cm). Thickness – 38µm.
Composition	Polyethylene bag equipped with ties made of recycled materials. 100% recycled super-absorbent pad composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. No latex and no phthalates. Packaging made of recycled materials.
Shelf life	No expiry date.
Efficiency	Absorption of 700 ml of liquids within 30 seconds.
Manufacturing process	Heat welding with insertion of sliding ties.
Use	For any person, men or women, who needs a bedpan and/or commode chair whether it's in an institution or at home. Particularly recommended for isolation case.
Instructions for use	<ol style="list-style-type: none"> 1. Open the packet and take out a roll of liners. 2. Unroll the first liner and tear it along the cutting line. 3. Open the liner and verify the presence of the super absorbent pad at the bottom of the liner. 4. Place the bag on the Hy21® bedpan support, the bedpan or the commode chair. 5. Let the patient relieve himself/herself into the liner. 6. Wait for the gelification of the liquids (± 700 ml within seconds). 7. Close the liner, tie it and discard. <p>Refer to the usage instructions for more detail.</p>
Precautions for use	Single-use product. Do not reuse. Keep away from children.
Disposal	Dispose of product in regular waste or according to the protocol of the institution. Don't dispose in toilet.
Benefits	Protection for Hy21® bedpan supports, standard bedpans or commode chairs. Increases hygiene, saves time, reduces odors, simple and risk-free disposal. Limits the spread of pathogens. Single-use product.
Identification	Each product is identified with a lot number.
Quality	Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: <i>Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.</i>
Regulatory	FDA: class I medical device exempt from 501(k). CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.
Risks/Safety	Slight risk of ocular or respiratory irritation in case of extended contact with the super-absorbent pad. Do not use the pad as a wipe on the skin, or put in diapers and incontinence underwear. Safety data sheet is available upon request.

**Handling/Storage
Packaging**

Unit	Box of 20 bedpan/commode liners	20 x 11.4 x 11.4 cm / 0.94 kg 8 x 4.5 x 4.5 in / 2.07 lbs
Case	20 boxes of 20 liners (400)	60 x 27 x 42 cm / 20.6 kg 23.5 x 10.75 x 23.5 inches / 45.5 lbs
Pallet	20 cases per pallet or 400 boxes of 20 liners	126 x 102 x 121 cm / 435 kg 49.5 x 40 x 47.75 inches / 960 lbs

