




Description	<p>Hygienic Cover® with a super-absorbent pad for urinal support Absorption capacity : 700 ml and more of liquid Cover dimensions : 17" x 6.3" (43 x 16 cm) No expiry date Eco-friendly product Made in Canada </p>
Item Number	PG1020
Presentation	This Hygienic cover® is a urinal bag containing a super-absorbent pad which solidifies bodily fluids and reduces odors. It is used in association with the Hy21® urinal support.
Cover dimensions	Length: 17" (43 cm). Width: 6.3" (16 cm). Thickness: 38µm.
Composition	Polyethylene bag equipped with ties. 100% recycled super-absorbent pad composed of a non-woven support with a sulfate base coupled with sodium acrylate based co-polymers. Packaging made of recycled materials. No latex and no phthalates.
Shelf life	No expiry date.
Efficiency	Absorption of 700 ml of liquids in 30 seconds.
Manufacturing process	Heat welding of plastic film with insertion of ties.
Use	For any person, men or women, who does not have access to traditional toilets or has difficulty using them, whether in a health care institution, at home or during a trip. Particularly recommended for isolation wards, maternity wards, emergency services, oncology and radiology.
Instructions for use	<ol style="list-style-type: none"> 1. Open the packet and take out a bag. 2. Open the bag and verify the presence of the super absorbent pad at the bottom of the bag. 3. Place the bag on the urinal support. 4. Urinate into the bag. 5. Wait for the gelification of the fluids (± 600 ml within seconds). 6. Close the bag, tie it and discard in the appropriate location. <p>Refer to the usage instructions for more detail.</p>
Precautions for use	Single-use product. Do not reuse. Keep away from children.
Disposal	Dispose of product in regular waste or according to the protocol of the institution. Don't dispose of in toilet.
Benefits	Protection for urinal support. Increases hygiene, disposable product, saves time, reduces odors, simple and risk-free disposal.
Identification	Each product is identified with a lot number.
Quality	Our bags are tested using the methodology and acceptance criteria of standard NF-X30-501: Packaging for medicinal care waste – Bags for soft infections risk-generating waste – Specifications and tests methods.
Regulatory	FDA: Class I medical device exempt from 501(k). CE marking: Class I medical device according to the rules of annex IX of European Council Directive 93/42/EEC.
Risks/Safety	Very slight risk of ocular or respiratory irritation in case of extended contact with the super-absorbent pad. Do not use the pad as a wipe on the skin, or put in diapers and incontinence underwear. Safety data sheet is available upon request.
Handling/Storage	Store in a clean and dry place, away from sun.

Packaging	Unit	Box of 20 urinal Hygienic Covers®	25 x 15 x 5 cm / 0.5 kg 9.8 x 6 x 2 in / 1.1 lb
	Case	20 boxes of 20 urinal bags (400 Hygienic Covers®)	60 x 27 x 42 cm / 10.8 kg 23.5 x 10.75 x 23.5 in / 24 lb
	Pallet	20 cases per pallet or 400 boxes of 20 urinal Hygienic Covers® (8000 Hygienic Covers®)	126 x 102 x 121 cm / 240 kg 49.5 x 40 x 47.75 in / 530 lb

